


RADIATION ATTENUATION APRONS

APRON CARE INSTRUCTIONS

APRON STORAGE

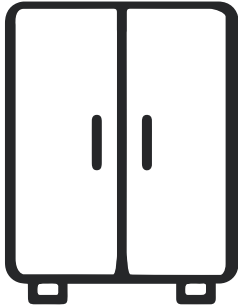

When you receive your apron, please store it properly to avoid any unnecessary damage. Best places to store your apron: Valet racks, peg racks, apron hangers and closets. If a hanger is not available, lay flat on the ground.

DO NOT: Please do not fold the aprons for an extended period of time. This will increase the changes of the inner material creasing or cracking. Keep it out of direct sunlight or prolonged expose to high heat.

AFTER EACH USE



It is important to establish a consistent, deep-cleaning schedule to mitigate the risk of pathogen transmission. Please clean the apron immediately after ease use. You can clean most dirt and stains (blood, bodily fluids, barium contrast media) using designated apron cleaning solutions (**PDF Sani-cloth AF3 Germicidal Disposable Wipes**) or mild household soaps in diluted form. To apply cleaning fluid, please use a cloth or wipe and always use cold water. **Please Note:** If you're dealing with biohazardous materials, we recommend using an additional barrier (such as SMS gowns) to protect the apron.

DO NOT: Do not use petroleum-based cleaning solvents or solutions containing bleach. Do not machine wash or dry your apron. Do not use hot or warm water. Do not iron or autoclave.

DISPOSAL

You must dispose of lead garments by complying with local, state, federal and/or international regulations where applicable.

